

SET® and rainbow® Sensor & Cable Compatibility

Masimo Technology ▶		rainbow SET®		SET®		
Patient Cable ▶		RD rainbow SET®	rainbow® RC	RD SET®	M-LNC	Red LNC/LNC
rainbow® Sensor ▼		Connector ▶		RD15	M15	DB9
RD rainbow SET-2 Disposables		Y	N	Y	N	N
RD rainbow™ 4λ Disposables		Y	N	Y	N	N
rainbow® Disposables R		N	Y	N	Y	N
rainbow® Disposables R1		N	Y	N	Y	N
rainbow ReSposable® R2		N	Y	N	Y	N
rainbow® SpHb® Spot-check Reusables		N	Y	N	N	N
rainbow® SpCO® Reusables		N	Y	N	Y	N
SET® Sensor ▼						
RD SET Disposables & Reusables		Y	N	Y	N	N
M-LNCS Disposables & Reusables		N	Y	N	Y	N
LNCS Disposables & Reusables		N	N	N	N	Y

Minimum Software Compatibility Requirement[†]

RD rainbow™/RD SET® Sensor ▼	Patient Cable ▶	RD rainbow SET®					RD SET®			
	Parameter ▶	SpHb*	SpMet*	SpCO*	RRp*	PVi*	SET*	RRp	PVi	SET*
RD rainbow SET-2 Disposables		7.4	7.1		7.9	4.8	No Min	7.9	4.8	No Min
RD rainbow™ 4λ Disposables					7.9	4.8	No Min	7.9	4.8	No Min
RD SET Disposables & Reusables					7.9	4.8	No Min	7.9	4.8	No Min

rainbow®/M-LNCS® Sensor ▼	Patient Cable ▶	rainbow® RC					M-LNC			
	Parameter ▶	SpHb	SpMet	SpCO	RRp	PVi	SET*	RRp	PVi	SET*
rainbow® Disposables R			7.1	7.1	7.9	4.8	No Min	7.9	4.8	No Min
rainbow® Disposables R1		7.4	7.1		7.9	4.8	No Min	7.9	4.8	No Min
rainbow ReSposable® R2		7.4	7.1		7.9	4.8	No Min	7.9	4.8	No Min
rainbow® SpHb® Spot-check Reusables*		7.4	7.4		7.9	7.4	7.4			
rainbow® SpCO® Reusables			7.1	7.1	7.9	4.8	No Min	7.9	4.8	No Min
M-LNCS Disposables & Reusables					7.9	4.8	No Min	7.9	4.8	No Min

LNCS® Sensor ▼	Patient Cable ▶	Red LNC					LNC			
	Parameter ▶	SpHb	SpMet	SpCO	RRp	PVi	SET*	RRp	PVi	SET*
LNCS Disposables & Reusables					7.9	4.8	No Min	7.9	4.8	No Min

[†] Masimo technology board software, sensor, and sensor cable combinations have been validated and tested for compliance per ISO 80601-2-61..

* Device must have SpHb parameter enabled.

RD rainbow 4λ Sensors are not licensed for sale in Canada.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, and precautions.

Masimo Americas
tel 1-877-452-7455
info-america@masimo.com

Masimo International
tel +41-32-720-1111
info-international@masimo.com

